

Suggested storage condition of our LiFePO4 batteries

Our powered Workstations on Wheels (WOWs) use lithium-ion (LiFePO4) batteries. The guidance below relates to the following carts:



Evie



Genna



VersaLite
EPR



VersaLite
MedDrawer



VersaLite
ePMA



MedRound

If your powered carts are not going to be used for several months, we recommend that you speak to a member of our customer service team and arrange storage decommissioning, which will maintain the battery warranty.

Our power systems are not designed for storage, so this recommendation is based on the experience of both Parity Medical and the battery manufacturer.

If you chose not to do this, please ensure the following conditions are met:



The battery should be stored in a well-ventilated area, away from direct sunlight and water. The temperature should not drop below 10°C or exceed 30°C. \$



The battery charge level is between 30% and 50% and topped up every 3 months to prevent over-discharge. \$



If batteries are left for an extended period of time without charge, it will result in permanent damage to the battery. If the battery does become fully discharged, charge it immediately. \$



Don't leave batteries unattended without proper charging.

Please contact Parity Medical's customer service team on +44 (0)151 343 0500 (option 2) or email us at customerservice@paritymedical.com for further assistance.